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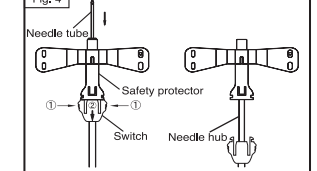
Fig. 5


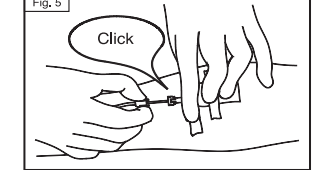
Fig. 6

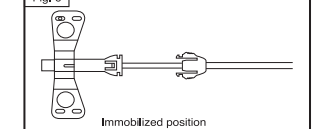NIPRO
NIPRO-VIETNAM CO., LTD
Lot 1-B-3, D8 Road, Tang Nhon Phu Ward,
700000, Ho Chi Minh City, VIETNAM

[Imported and Distributed by]
NIPRO MEDICAL PHILIPPINES CORPORATION
29F, Unit C&D, Menarco Tower, 32nd St., Bonifacio Global City,
Fort Bonifacio, Taguig City, Metro Manila

[Distributed by]
PT.SINAR RODA UTAMA
Duta Merlin Blok C No.58
Jl. Gajah Mada No.3-5
Jakarta - Indonesia

[Authorized Representative in Malaysia]
NIPRO MALAYSIA SDN. BHD.
B-3-2, the Ascent Paradigm, No.1 Jalan SS7/26A,
Kelana Jaya, 47301 Petaling Jaya, Selangor, Malaysia
Telephone: +603-7622 1190
General email address: office@nipro.com.my

MADE IN VIETNAM

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SAFETOUCH™ PSV SET SAFETY WINGED NEEDLE SET

<Instructions for Use>

Prior to use, please read the Instructions for Use carefully.

Indications

The device is indicated for insertion into a patient's vascular system or subcutaneous tissue to administer medicinal products. It is intended for use by healthcare professionals with specialized knowledge, including doctors, medical technicians, and nurses. Additionally, patients or their families may use the device under the direction of healthcare professionals.

Contraindications

- Do not re-use.
- Do not use the device for high-pressure infusion, such as for angiography with an injector or a similar tool.

Shape and Structure

This device consists of a needle tube, a needle hub, wings, a tube, a safety protector, a switch and a luer connector.

For safe disposal after use, the safety protector has a safety mechanism that protects and covers the used needle tube.

Operating Principle

After using the product, pinch both ends of the switch to unlock the safety mechanism. Subsequently, the safety protector will make a clicking sound when the needle tube is removed from the blood vessel while holding the wings. The needle tube then retracts into the safety protector shielding the needle tip.

Clinical Benefits

It is possible to treat and prevent disease by administering medicinal products through infusion when the device is connected with a syringe, infusion set, etc. Additionally, blood samples may be collected for inspection.

Cautions

1. Cautions related to usage, etc.
 - 1) Handle the device carefully to prevent needlestick injuries.
 - 2) When removing the needle cap, pull it straight off to avoid touching the needle tip. Deformation of the needle tip may occur, making puncturing more difficult. (Fig. 1)
 - 3) Do not allow medicinal product to adhere to the luer cap, as this may cause the connections to become loose.
 - 4) If any abnormalities, such as fluid leakage from the connecting part, are found during priming, do not use the device.
 - 5) Use the device immediately after opening the package.
2. General
 - 1) When opening the packaging, take care to prevent the film from catching on the device, especially the film covering the needle cap. Avoid exerting excessive force to prevent bending of the needle tube while opening the packaging.
 - 2) Regularly inspect the device for any signs of damage, loose connections, or fluid leakage during use.
 - 3) Exercise caution when administering a fat emulsion or any medicinal product containing fat emulsion, oils such as castor oil, or any medicinal product containing solubilizing agents like surface-active agents or alcohol, or when using a disinfectant containing alcohol. The medicinal product may cause cracks in the luer connector and needle hub, resulting in leakage of blood or medicinal product, air inclusion, etc. Inaccurate dosage - caused by cracks in the luer connector and needle hub - can occur due to the use of fluids containing fat emulsion, oils such as castor oil, or solubilizing agents such as surface-active agent or alcohol, and tightening or retightening with excessive force. Pay particular attention to dose-dependent drugs since inaccuracies can have harmful consequences on patients.
 - 4) Immediately replace the device with a new one if any cracks are discovered on the connecting parts or elsewhere.
 - 5) Handle the tube carefully - avoid using forceps or similar tools. To prevent the risk of fluid leakage, air inclusion, or breakage, take care not to damage the tube with sharp instruments such as the tip of a needle or scissors.
 - 6) To prevent the tube from breaking, kinking, or being damaged, ensure that the tube is not trapped under the patient's body.
 - 7) To prevent the risk of disconnection, damage, or elongation of the tube, do not pull out, push in, or bend the area connecting the tube and the luer connector with excessive force.
 - 8) To prevent the risk of bending or breakage of the needle tube, or damage to the needle hub, avoid applying excessive force on the needle tube.
 - 9) To prevent the risk of the luer connector breaking or disconnecting, which could result in fluid leakage or air inclusion, refrain from excessively tightening the luer connector.
 - 10) Secure the wings firmly. When inserting the needle into a vein or using the safety mechanism to remove the device, be cautious as the needle tube may misalign or damage the walls of blood vessels.
 - 11) Start infusing medicinal products immediately after priming is completed.
 - 12) If puncturing a rubber button or similar surface, secure the needle at the puncture site. An excessive load may cause the needle tube to bend or break, or damage the needle hub.
 - 13) Avoid holding the switch while puncturing as the switch may unlock, resulting in an inability to puncture.
 - 14) Prevent alcohol or other medicinal products from adhering near the base of the needle tube. This can cause leakage between the safety protector and the needle hub, leading to damage of the needle hub.
 - 15) Only unlock the safety mechanism when you are ready to remove the needle from the patient.
 - 16) When unlocking the switch to store the needle tube, fully unlock the lock and pull straight without applying lateral force. Pulling it out at an angle could result in excessive pressure, potentially causing breakage.
 - 17) After using the safety mechanism, avoid bending the device between the switch and needle hub to prevent accidental sharps injuries or blood leakage.
 - 18) If an abnormality is found that prevents unlocking of the switch lock, remove the device without using the safety mechanism. Promptly dispose of the device in a disposal container, taking care to avoid sharps injuries from the needle tube.
 - 19) Do not attempt to unlock the safety mechanism after it has been used, as there is a risk of the needle tube sticking out.
 - 20) Dispose of the device in an approved biohazard container as per facility protocol.
 - 21) Report all serious incidents to both the manufacturer and the competent authority of your State.
 - 22) If the device is exposed to abnormal conditions (for example, high temperature and humidity) or if it is unintentionally opened before use, do not use the device.
 - 23) Do not use the device if it is damaged or shows any other abnormalities.
 - 24) Reusing the device may result in infection due to contamination.

Instructions for Use

1. Instructions for infusion

- 1) Open the package and remove the device.
- 2) Remove the luer cap from the luer connector, and firmly connect the infusion set or the nozzle of a syringe to the luer connector. Ensure that the safety protector is securely locked into the switch. (Fig. 2)
- 3) Fill the tube with the medicinal product, allowing it to flow out from the tip of the needle tube, and remove all air from the tube.
- 4) Disinfect the puncture site.
- 5) Hold the wings and remove the needle cap from the needle tube by pulling it straight off. (Fig. 1) While holding the wings, insert the needle into a vein or subcutaneous tissue.
- 6) After inserting the needle into a vein, confirm that blood flows into the tube, and then secure the wings with tape. Also, ensure that the tape is not attached to the switch. (Fig. 3)

2. Instructions for collecting blood








- 1) Open the package and remove the device. Connect the luer adapter or the nozzle of a syringe to the luer connector firmly.
- 2) Apply a blood pressure cuff and disinfect the puncture site.
- 3) Hold the wings together and remove the needle cap from the needle tube by pulling it straight off. (Fig. 1) Insert the needle into the puncture site.
- 4) Collect the required amount of blood using a vacuum blood collection tube attached to the luer adapter or syringe.
- 5) After collecting blood, remove the blood pressure cuff. Activate the safety mechanism and remove the device while applying pressure to stop bleeding at the puncture site.
- 6) Safely dispose of the device after use.

3. Instructions for using the safety mechanism

- 1) After using the device, release the lock by pressing both sides of the switch inwards. (① in Fig. 4)
- 2) Slide the switch backward (② in Fig. 4) until it becomes completely locked with an audible and tactile click. (Fig. 5)
- 3) Confirm that the switch is securely immobilized against the safety protector. (Fig. 6)
- 4) Apply pressure to the puncture site if necessary.

Connection with other devices

The device is connected to syringe, luer adapter or an infusion set, etc. through the luer connector. The luer connector is compliant with ISO 80369-7.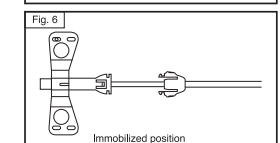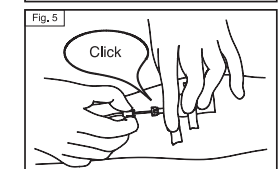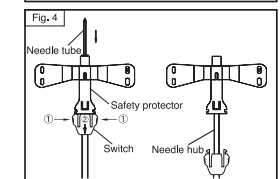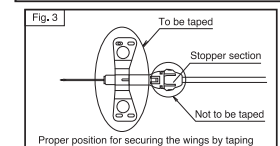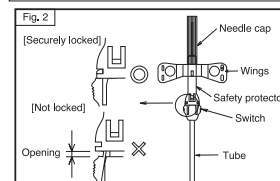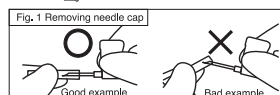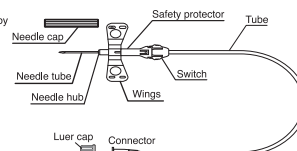

NIPRO VIETNAM CO., LTD
Lot I-8-3, D8 Road, Tang Nhon Phu Ward,
700000, Ho Chi Minh City, VIETNAM

[Imported and Distributed by]
NIPRO MEDICAL PHILIPPINES CORPORATION
29F, Unit C&D, Menarco Tower, 32nd St., Bonifacio Global City,
Fort Bonifacio, Taguig City, Metro Manila

[Distributed by]
PT. Sinar Roda Utama
Duta Merlin Blok C No.58
Jl. Gajah Mada No.3-5
Jakarta - Indonesia

[Authorized Representative in Malaysia]
NIPRO MALAYSIA SDN. BHD.
B-3-2, the Ascent Paradigm, No1 Jalan SS7/26A,
Kelana Jaya, 47301 Petaling Jaya, Selangor, Malaysia
Telephone: +603-7622 1190
General email address: office@nipro.com.my

MADE IN VIETNAM